You must include the following items in your FedEx package:

Test Requisition Form

Insulated foil bag

Ice pack

Biohazard bag

Two sample tubes

Please retain your tracking number (packaged with your FedEx bag) to ensure safe delivery of your package.

Your package may be dropped off at any FedEx location.  
Find a location or schedule a pickup by visiting [www.fedex.com](http://www.fedex.com) or calling 1-800-GO-FEDEX

For additional questions, please call us  
(732) 542-3200  
between 9 am - 5 pm EST, Monday through Friday  
[www.yourgotolab.com](http://www.yourgotolab.com)

Before you begin, write your name and date of birth on the labels on both tubes and the plastic specimen bag.

Be certain to freeze your ice pack as well.

In preparation, put on the included gloves.

Open the white paper collection bowl and follow the instructions to install it on your toilet. Have a bowel movement into the paper bowl.

Blue Cap Tube: Use the scoop attached to the lid to scoop feces into the tube to the **15 ml fill line**. Green Cap Tube: Use scoop to fill to the **fill line**.

Place both tubes into the plastic specimen bag.

Take the ice pack and the specimen bag and place them together into the insulated foil bag. Seal the bag.

Check the information, including your insurance, for accuracy. The rest of the form should have been completed by your doctor.

Pack the sealed foil bag and the completed form into your FedEx bag.

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